

ACQ426 / 250 ml Actiomedic Ph Neutralized Solution

It is used to neutralize the eye for two minutes in case of eye injuries caused by acid and alkali exposure to the eye (excluding lime and cement containing calcium substances). After rinsing eye with PH Neutral (ACQ26), it is recommended to rinse with sterile sodium chloride solution (ACQ25) until medical support.



Technical Specifications

Content	Actiomedic Ph Neutralized Solution
Color	Transparent, colorless
Form	Liquid
Scent	Unscented
Package Quantity	10 Pieces

STARLINE

USAGE



The arrow mark on the eyewash cap is turned in the direction of the arrow, the cap is tightened with the thumb and forefinger, the eyewash cap rests against the eyecup when it is in this position and the fingers are released so that the eyelids will remain open while the eyewash is applied, tilting the head to the floor lightly squeezes the eyewash bottle and initiate contact with the eye, the excess liquid will flow down the drain holes in the bottle cap, the remaining eye solution is applied to the eye by inverting and squeezing the bottle, each eyewash bottle covers a first aid, then not used again. Eyewashes are single use only.

Important! In all types of damage, the eye wash liquid should be applied softly and evenly to the eye. Press gently. Always consult a doctor and continue to wash during transfer.

Required Applications

250 ml eye fluids are applied within a period of 5 minutes. Each eye fluid is single-use only. Repeated use is not appropriate. It is mandatory to use it from the opening. After 15 minutes, rinse with plenty of water can be carried out normally.

Storage

Please keep unopened original packaging, in a temperature range of 5-25 C degrees and in a dry environment. Protect the product from sunlight and UV rays to avoid damage.

Areas Of Usage



Agriculture



Construction



Dyeing and Printing



Automobile



Cleaning



Chemicals

It is used to ejaculation of foreign substances, chemical and derivatives of acid-caustic-alkalis and their contact with skin. It is used to prevent small and invisible particles that may damage eye such as dust, dirt, metal and wood. It is used to wash eye in case of contact of chemical liquids and particulates.

STARLINE

STATIONS

Two eye shower stations sold in groups are listed below.



CPS023 - Eye Shower Station
(2Pieces. ACQ425 + 1Piece Mirror)

Mounting materials
1 piece wall board
1 piece mounted on the panel mirror
1 piece set of mounting material
(dowel-screw)



CPS024 - Eye Shower Station
(1Piece ACQ425 + 1Piece ACQ426 + 1Piece Mirror)

Sipariş Bilgisi

MODEL	Size	Barcode	Box Quantity	Box Dimesion	Box Weight
CPS023 Eye Shower St.		8680907926829	1 Piece	33 x 43 x 9cm	1.80kg.
CPS024 Eye Shower St.		8680907926775	1 Piece	33 x 14 x 9cm	1.60kg.
ACQ425 / 500ml		8680907926744	10 Pieces	14 x 33 x 28cm	6.20kg.
ACQ426 / 250ml		8680907926768	10 Pieces	14 x 32 x 21cm	3.40kg.